



Tender-Trode Plus™

Neonatal and Micro-Premie Leads

PROPER USE PROCEDURES

Prior to Use

- Skin must be clean and dry per hospital protocol for best results
- Moisture, oils or lotions adversely affect the adhesion of hydrocolloid
- No skin prep is required or recommended
- **DO NOT USE ALCOHOL**

Application

- Remove electrodes from the pouch
- Gently warm for 30 to 45 seconds for best application
- **Hold in place for at least 60 seconds to ensure proper adhesion**
- Heat will increase the bond between the electrode and the patient

Removal

- **Saturate surface of electrode with water, gently peel away from skin as you swab with water**
- Soap or mineral oil may also be used for removal

Cautions

- Patient must be under professional supervision
- Discontinue use immediately if skin irritation occurs
- May be used for X-Ray and Fluoroscopic Procedures
- Not for use in MRI



Do Not Re-use




Latex Free

LOT

Lot Number of Product

Rx ONLY

CAUTION: Federal (US A) law restricts this device to sale by or on the order of a physician.

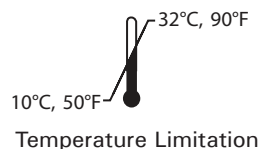

Temperature Limitation



Date of Expiration
(24 months from date of mfg.)